Production Standards
Normes de Production
Produktion Standard
Productie Standaarden

EN

At Nimco Made4you we combine traditional methods and procedures of orthopaedic shoe making, with the best and most modern materials. Our commitment to quality and innovation permits us to produce beautiful shoes that support and care for special feet.

In the following pages you'll find our internal standards, our predefined shoe construction method. Help us fill your wishes, in the best way, by paying attention to these principles and be sure to communicate what alterations you feel necessary.

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Technical Orthopedic Footwear Production

NM4Y reserves the right to change the specifications, rules and general conditions on this document, notifying its clients by email and / or website.

I Client Order Form

The client should use the order forms available in NM4Y's website. On these order forms all relevant specifications and information for the order execution should be filled in. If the client uses his/her own form, he/she should inform NM4Y whenever there are changes.

The order form should be accurate and include all necessary information in a clear and complete way.

NM4Y commits itself to follow the instructions described in the client's order form, as well as protocol. The written information included in the order form is considered as a priority over other information eventually attached/sent and not mentioned in the order form at hand.

If the information provided by the customer is incomplete but enough to fulfill the order, NM4Y reserves the right to interpret and execute according to its own specifications without being obliged to previously notify the client about, namely: missing color distribution, stitches, bottoms, decorations, leg measurements, height, velcro straps, etc.

Throughout the production process, situations may occur that force NM4Y to unlast the pairs, such as: wooden heels, stitched welt construction, lining quality check (with high counters), among others. Therefore, client's uncut lasts are to be cut by NM4Y for quality, safety and employees' physical integrity matters.

In case of contradictory information, the data in the order form prevails.

If the client wants the sent patterns to be used as baseline for the model, he/she should select the option "check pattern". If the client wants the patterns to be used without changes, he/she should select the option "pattern included". In this case, the pattern has priority over photographs/drawings.

In models from NM4Y catalogue, NM4Y executes the model including: embroideries, laser and decoration features, unless stated otherwise on the order form.

In models from other catalogues, NM4Y will only execute embroideries, laser and decoration features when requested on the order form. NM4Y will not execute them even if they are included in; photographs, patterns, moulds or previous orders.

II Materials

When the client sends materials, these must be properly labelled with the process number/name and mentioned in the order form. Whenever applicable, the client should mention which material's side to use.

NM4Y guarantees quality and availability of all materials provided in its own catalogue/portfolio.

III Special Requests

Special requests, which are not included in NM4Y portfolio, are only executed if the client specifically mentions so on the order form. This request will always depend on the availability of local suppliers and will add costs and production time to the order.

IV Models Catalogue NM4Y MCO

In models from its own catalogue, NM4Y follows the model unless stated otherwise on the order form and/or technically impossible.

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EN

Leg Measurements

If the measurements given in the order form are inferior to the lasts' perimeter, the modelling will follow the lasts' measurements.

Instep Line

The length of the last is measured from the highest point of the heel to the tip toe, as in the image: the instep line corresponds to $\frac{2}{3}$ of the length of last. The distance between the instep and the front is measured from the instep to the inferior border of the last.

(01)

Koker:

If the client does not mention the upper's total height, it will be considered the height of koker + 10 mm. If the client specifies the upper's height, this should be followed, even if the koker overlaps the upper.

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EN

Leaning

The quarter top part of the boot leans 4 to 5 degrees: (O2)

Loop strap

Loop straps are reinforced and underlaid.

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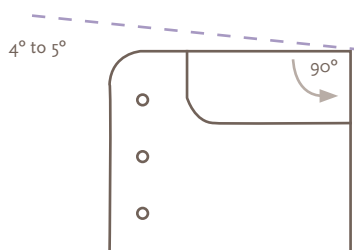
EN

Height

Shoes - the tongue ends 5 to 10 mm above the quarters.
Boots - the tongue ends 10 to 15 mm above the quarters.

The tongue velcro is placed 5 mm from the edge of the tongue and 10mm from the edge of the facings, in the quarters.

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EN

Anti-slip Heel

Boots – only applicable when requested by client.
Shoes – always applicable, except with fabric lining.

Notice: when the client orders anti-slip heel with textile or fur lining and doesn't mention the material, a cow lining anti-slip heel in the closest possible color will be used.

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shoes – 5 mm
boots – 10 mm
sport shoes – 20 mm
(01)

Distance

Eyelets/hooks should be placed at **15 mm** from the edge. (02)

Rings and double hooks are placed at **20 mm** from the edge.
(03)

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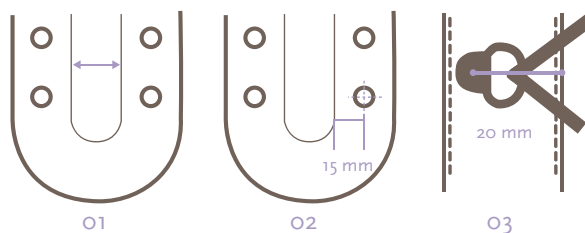
Ankle Orthosis / Embrases / Peronäusschiene / Omsluiting

EN

Ankle orthosis are produced as follows:

- A - Neoprene on the inside
- B - Always with collar (create collar if necessary)

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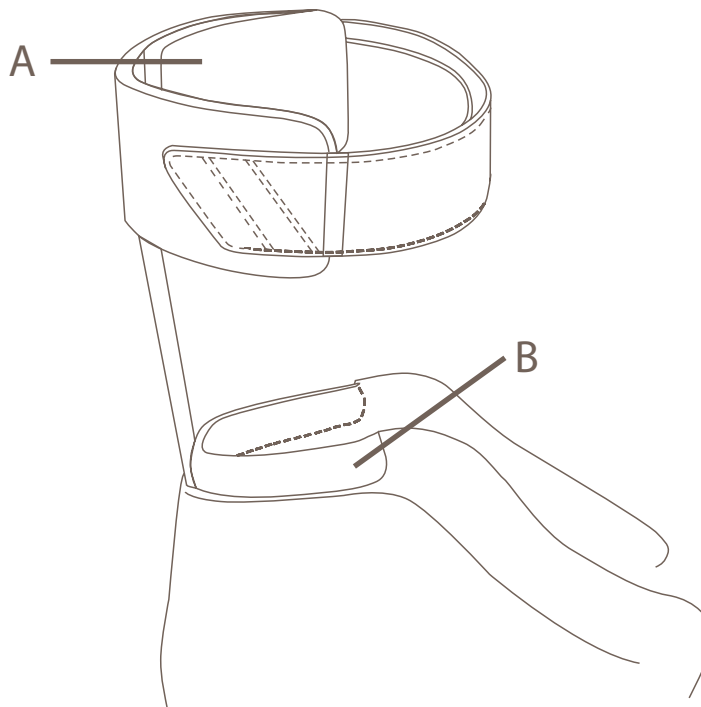
EN

The tongue velcro is placed 5 mm from the edge of the tongue and 10mm from the edge of the facings, in the quarters. (01)

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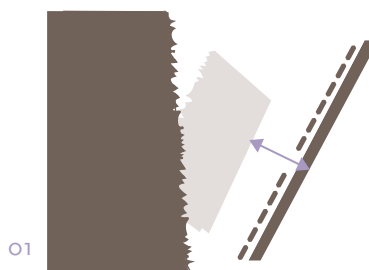
EN

Trimming

The stitch /seam is done between leather and lining. In derby models the trimming area is reinforced.

Eyelets and hooks

The eyelets and hooks have the following sizes:

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EN

Velcro

Velcros have two cutting options and roller.
The velcro order is the following:

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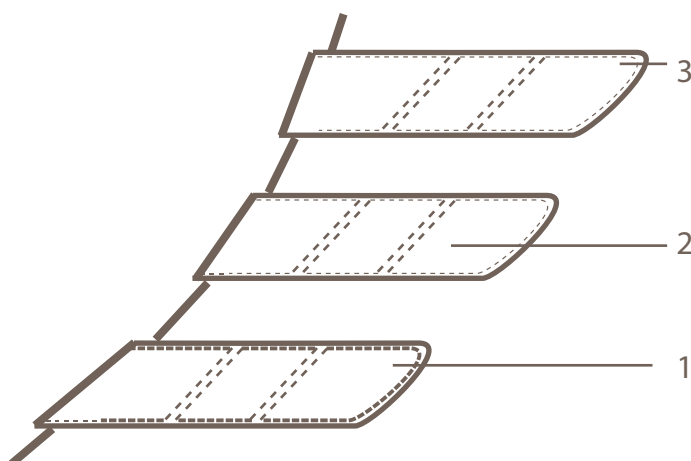
Closure - Facings / Fermeture / Verschluss / Sluiting

EN

Velcro strap length is between 210-230 mm: rough side is below with 90 mm while the remaining is the soft side (01)
A - Rough B - Soft

Velcro stitches are done at 2 mm from the edge. (02)

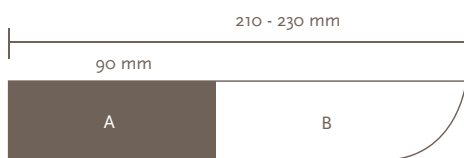
Velcro straps are 5 to 15 mm apart from the rough side, as the following image. (03)

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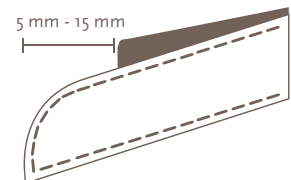
EN

Zipper

Functional zippers are in plastic and the lid has no lining. (01)

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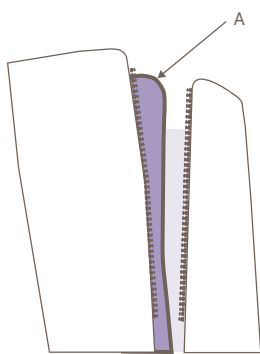
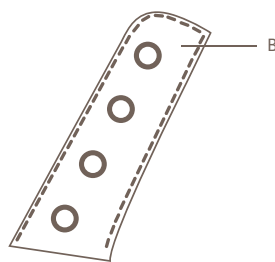
EN

Open toe models

In open toe models, we send leather skived down to 0,7 mm to cover the supplement.

Heels

The leather to cover the heel is the same color as the counter:

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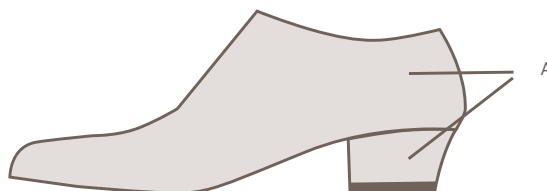
EN

The thread thickness is chosen according to the following table.

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Insole / Premières / Brandsohle / Binnenzool

EN

The insoles' material is always Texon.

Carbon insole material is cut out of a sheet.

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Toe Cap / Bouts Durs / Vorderkappe / Neuzen

EN

The toe cap's measurements are the following:

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EN

All the shoes and boots have a side reinforcement:

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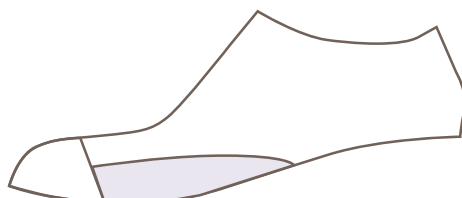
Stiffeners - Counters / Contreforts / Hinterkappe / Contreforts

EN

The stiffener's material is Rhenoflex 3202 (1.8mm).
In shoes with padded collar, the stiffener should go up to the bottom border of the padding.
In shoes without padded collar, the counter should stay 3 to 5mm from the top:

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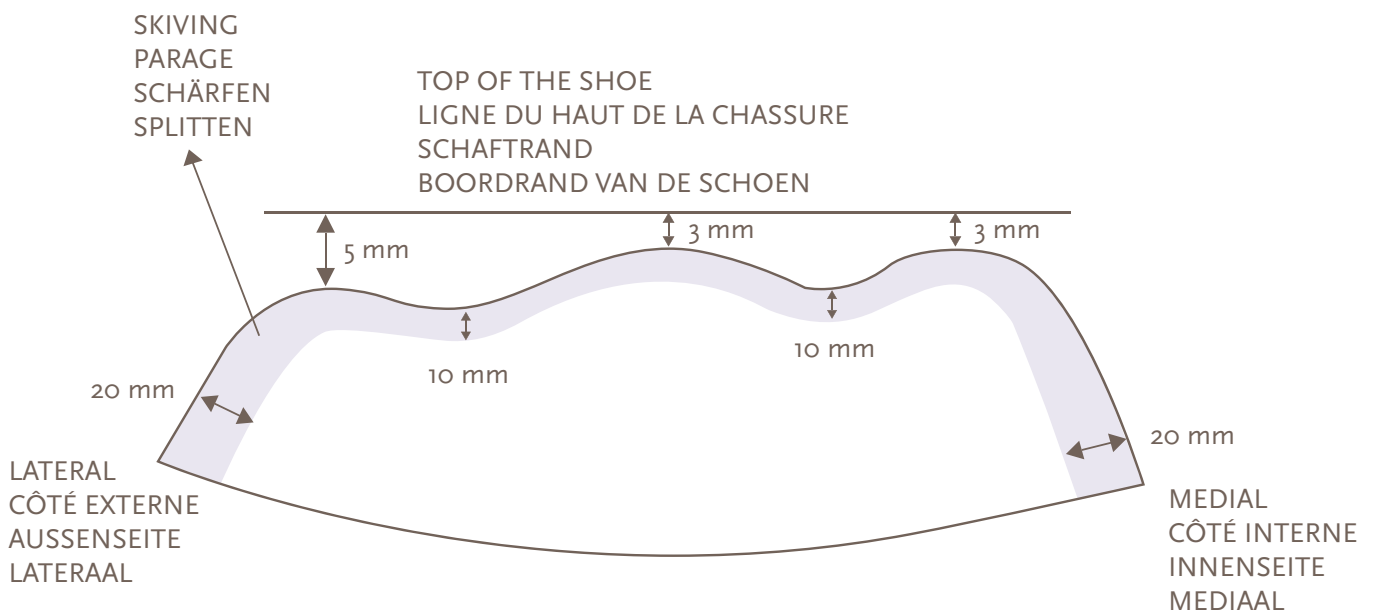
Stiffeners - Counters / Contreforts / Hinterkappe / Contreforts

EN

The different types of stiffeners are:

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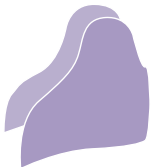
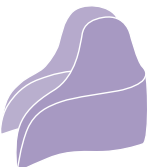
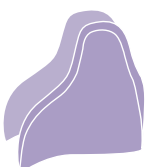
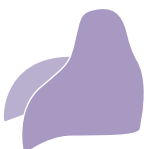
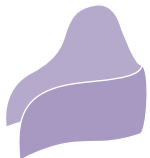
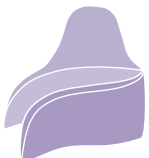
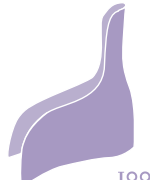
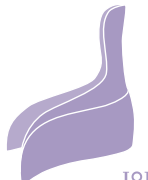
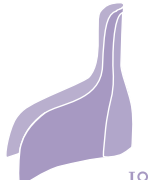
EN

Heights

The height of welts excluding heel is the following:

- Leather - 12 mm high on the lateral side;
- Rubber/EVA - 15 mm high on the lateral side;

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EN

Block soles (ex: 2810/2600/2640)

On block soles we apply a wedge to compensate for the heel height:

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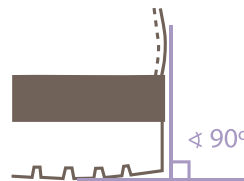
Heel rounding is 5 mm.

Height

The heel is measured in the back as follows:

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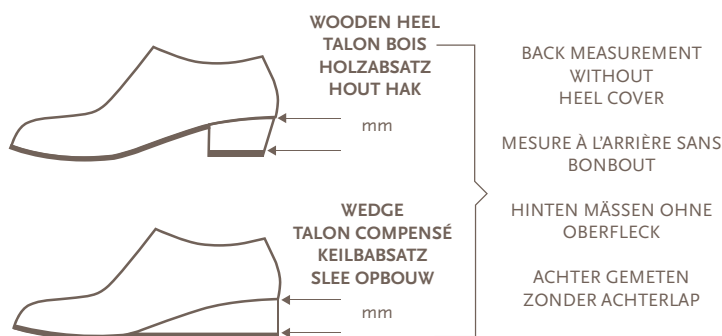
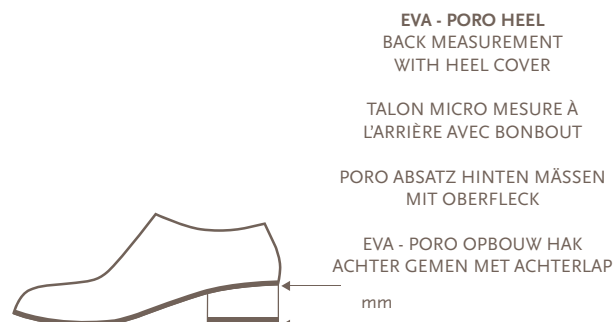
EN

Heel tip

On wooden heels we use nylon tips in the colors: black, brown and natural colors. With other colors we use matching rubber tips.

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Packaging / Emballage / Verpackung / Verpakking

EN

The shoes are wrapped in aircup.
All client's material (including file, patterns and moulds) is sent.

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Nimco Made4You, B.V.