

Moveo S.r.l.

Via Monsignor G. Fortin, 38
35128, Padova, Italy
+39 049 26144 27
VAT IT 05236760285
info@moveowalks.com
moveowalks.com



ExoBand Washing and Maintenance

Hand Washing

1. **Preparation:** Ensure the elastic components are removed from the thigh pieces as described in the "MAINTENANCE" section.
2. **Sponge and Detergent:** Use a soft sponge and a mild, neutral detergent. Avoid harsh detergents that could damage the device material.
3. **Cleaning:**
 - Lightly dampen the sponge with cold water.
 - Apply a small amount of detergent to the sponge.
 - Gently scrub all surfaces of the device, avoiding complete immersion in water.
4. **Rinsing:** Clean with a fresh, damp sponge to remove any detergent residue.
5. **Drying:** Do not use a dryer. Air dry the device away from direct heat sources and sunlight.

Machine Washing

1. **Preparation:** Ensure the elastic components are removed from the thigh pieces as described in the "MAINTENANCE" section.
2. **Washing Machine Settings:**
 - **Temperature:** Select a wash cycle with a maximum temperature of 30°C. Do not wash at temperatures above 30°C.
 - **Cycle:** Use a delicate cycle to avoid damage to the device.
 - spin cycle is prohibited.
3. **Detergent:** Use a mild detergent suitable for technical fabrics.
4. **Drying:** Do not use a dryer. Air dry the device away from direct heat sources and sunlight.

Sanitizing with Disinfectant Spray for Technical Fabrics

1. **Choosing the Spray:** Use a disinfectant spray specific for technical fabrics, ensuring it is suitable for use on medical devices.
2. **Application:**
 - Spray evenly over the entire surface of the device.
 - Ensure the device is clean before applying the spray to ensure effective sanitization.

Moveo S.r.l.

Via Monsignor G. Fortin, 38
35128, Padova, Italy
+39 049 26144 27
VAT IT 05236760285
info@moveowalks.com
moveowalks.com



3. **Drying:** Allow to air dry. Avoid using the device until it is completely dry.

Additional Notes

- Do not use bleach or other harsh chemicals that could damage the device.
- spin cycle is prohibited.
- Regularly check the condition of the device to ensure there are no damages or signs of wear that could compromise its functionality.
- Do not iron.
- Always follow the manufacturer's specific instructions for the maintenance and care of the device.

By following these instructions, you will be able to keep the medical device clean, hygienic, and functioning optimally.

Extraordinary Maintenance

Extraordinary maintenance is required in case of breakages due to unforeseen accidents or inappropriate use of the medical device. The situations that may arise are entirely unpredictable and therefore it is not possible to describe appropriate intervention procedures. In case of need, consult the technical service of Moveo S.r.l. to receive instructions appropriate to the situation. All extraordinary interventions must be carried out by specialized and authorized personnel, otherwise the warranty will be voided.

For more information on maintenance, please read the user and instruction manual.